# Nourisil™ MD

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## [UK] Instructions for use

### Nourisil™ MD – a silicone gel for the prevention and treatment of new and old keloids and hypertrophic scars

#### What is Nourisil™ MD?

Nourisil™ MD is an ultra-light, transparent, self-drying silicone gel, which has been developed for the management and prevention of new and old keloids and hypertrophic scars, which result from general surgical procedures, wounds, trauma or burns. It contains a unique blend of silicones and vitamin E to maintain the skin's moisture balance, while improving the appearance of the scar. Nourisil™ MD helps to flatten, soften and smooth scars, relieve the itching and discomfort of the skin caused by scars and reduces associated pain and redness. Nourisil™ MD gel has an elegant, silky skin feel and is non-greasy. It can be easily applied to scars on all areas of the skin, including the face and joints. When used as directed, Nourisil™ MD dries to form a layer that hydrates and protects scars.

#### Intended use

Nourisil™ MD is used for the prevention and treatment of new and old keloids and hypertrophic scars, which result from general surgical procedures, wounds, trauma or burns. Nourisil™ MD can be applied after the wound has healed and the skin surface is intact. Nourisil™ MD is suitable for use on all skin types, including sensitive skin and on children. Nourisil™ MD is a medical device.

#### Directions for use

- Wash hands before and after use.
- Ensure the skin area is clean and dry.
- Remove cap from dispenser. Press the pump a few times to make the gel appear.
- Gently massage a small amount of Nourisil™ MD on the scar and allow to dry. If needed, carefully wipe away any excess with a tissue to avoid staining of clothing.
- Once dry, Nourisil™ MD may be covered with sun block or cosmetics.
- Apply Nourisil™ MD twice daily; once in the morning and once at night.

The recommended duration of treatment is 60-90 days. For larger or older scars, longer treatment may be required.

#### Caution

- Nourisil™ MD should not be applied to open wounds or third degree burn wounds.
- Nourisil™ MD should not be applied where dermatological conditions disrupt the integrity of the skin. In case of doubt, contact your doctor or pharmacist.
- Direct contact with mucous membranes and the eyes should be avoided.
- When additional topical treatment is used on the same skin area, please consult your doctor or pharmacist.
- Do not use if hypersensitive to one of the ingredients.
- If irritation occurs, use of the product should be discontinued immediately and a doctor or pharmacist should be consulted.
- For external use only.

#### Storage conditions

- Keep out of reach of children.
- Store between 4 and 25 °C.
- Do not use Nourisil™ MD more than 12 months after opening and not after the indicated expiry date (EXP).

#### Ingredients

Cyclopentasiloxane, Caprylyl Methicone, Tocopheryl Acetate, Dimethicone Crosspolymer, PEG-12 Dimethicone/PPG-20 Crosspolymer, Polyglyceryl-3 Diisostearate, Dimethicone.

#### Contents

30 g e

#### Further information

For more information, please consult your pharmacist, doctor or visit [fagron.com](http://fagron.com).

#### Manufacturer

Fagron BV, Lichtenauerlaan 182, 3062 ME Rotterdam, The Netherlands

#### Producer

Herkel B.V. Nobelweg 6, 3899 BN Zeewolde, The Netherlands

#### Distributor

Fagron BV, Lichtenauerlaan 182, 3062 ME Rotterdam, The Netherlands

[fagron.com](http://fagron.com)

This leaflet was last revised in 02/2020