Nourisil™ MD

[DA] Forholdsregler

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[2024-04-25]

[EN] Precautions

Nourisil™ MD helps to flatten, soften and smooth scars, relieve the itching and discomfort of the skin caused by scars and reduces associated pain and redness. Nourisil™ MD can be used for old and new scars.

Intended use

Nourisil™ MD is an ultra-light, transparent, self-drying silicone gel. When used as directed, Nourisil™ MD dries to form a layer that hydrates and protects scars. Nourisil™ MD is used for the prevention and treatment of keloids and hypertrophic scars, which result from general surgical procedures, wounds, trauma, burns or acne. Nourisil™ MD can be applied after the wound has healed and the skin surface is intact. Nourisil™ MD is suitable for use on all skin types, including sensitive skin and on children (from 1 year old). Nourisil™ MD is registered as a Medical Device.

Directions for use

- Wash hands before and after use.
- Ensure the skin area is clean and dry.
- In case of the dispenser: remove the cap and press the pump a few times to make the gel appear.
- Gently massage a small amount of Nourisil™ MD on the scar and allow it to dry.
- If needed, carefully wipe away any excess with a tissue to avoid staining of clothing.
- Once dry, Nourisil™ MD may be covered with sun block or cosmetics.
- Apply Nourisil™ MD twice daily; once in the morning and once at night.
- The recommended duration of treatment is 60-90 days.

For larger or older scars, longer treatment may be required.

Caution

- Nourisil™ MD should not be applied to open wounds or third degree burns.
- Direct contact with mucous membranes and the eyes should be avoided.
- When additional topical treatment is used on the same skin area, please consult your doctor or pharmacist.
- For external use only.
- Keep out of reach of children.
- Do not use after the indicated expiry date (EXP).

Contra-indications

- Stop using Nourisil™ MD if you get an allergic reaction.
- If irritation occurs, use of the product should be discontinued immediately and a doctor or pharmacist should be consulted.

Alleen

Storage conditions

Store between 4 and 25°C

Ingredients

Methicone, Dimethicone Crosspolymer, PEG-12 Dimethicone/PPG-20 Crosspolymer, Polyglyceryl-3 Diisostearate, Dimethicone.

Content

10g, 30g

Further information

In case of any serious incident, please inform the manufacturer and the national competent authority of the country where you reside. The cap of the bottle should be recycled as plastics and the pump should be disposed of with common household waste.

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